

# Alcohol free disinfectant wipes SEPTIWIPES

REF. GE-208-D100



ALCOHOL FREE



- < ALCOHOL FREE
- < PHENOL & ALDEHYDE FREE
- < VALIDATED ON ESAOTE, GE, PHILIPS, SONOSCAPE & CARESONO PROBES

Recommended for the cleaning and disinfection of non-invasive ultrasound probes by wet wiping.

CE  
0482

## DIRECTIONS FOR USE

Active on contact with the medical device, total activity in 5 minutes.  
Pull out a Septiwipes out of its packaging.  
Clean the medical device by wet wiping.  
Use a fresh wipe for each medical device.

EXCEPTIONS FOR USE : when treating unknown materials, carry out a trial upon a small surface.

STOCKING : After use close the capcarefully.

## EFFICACITY

- **Bactericidal** : EN 1040, EN 13727, EN 1276, EN 13697 - 5 min, assay on presoaked gauze : Enterococcus hirae, Pseudomonas aeruginosa, Staphylococcus aureus.
- **Yeasticidal** : EN 1275, EN 13624, EN 1650, EN 13697 - 5 min on Candida albicans.
- **Virucidal** : EN 14476 Vaccinia virus 30 sec; BVDV, Hepatiti C virus, Rotavirus, HIV 30 sec; SV40, Papovavirus, Polyomavirus, Influenta A/H1N1, H5/N1, Hepatiti B virus 2 min.

## SOLUTION COMPOSITION

0,25 g didecyldimethyl ammonium chloride  
0,5 g Quaternary ammonium compounds,  
benzyl-C12-16-alkyldimethyl-, chlorides.